

MEDI
DERMA-S

FACIAL SKIN PROTECTION FOR FFP3 MASKS OR EQUIVALENTS

The current COVID-19 situation has seen some healthcare practitioners noticing early signs of skin damage from tight-fitting masks. Our barrier products are effective in cases where irritation behind masks is caused by perspiration and friction, if used as per our guidelines. However, we **DO NOT** advocate the use of our barrier film products for the prevention of damage that may be caused solely as a result of direct pressure.

Prevention and Maintenance Normal Intact, Red/Discoloured Skin

MEDI DERMA-S Total Barrier Film Wipe and 1ml Sterile Applicator

- Ensure the skin is clean and dry before applying the barrier film
- Using the Wipe or Sterile Applicator, apply the film in a uniform coating to the area of the face to be protected; **ensuring the eyes are kept closed during application**
- Allow **at least 30 seconds** for the film to dry after application before applying the mask
- Apply mask as normal, following any instructions provided
- **Film application should be repeated at the start of every shift**



Superficial Face Wounds and Broken Skin

MEDI DERMA-S Total Barrier Film may be used underneath adhesive dressings to prevent further skin damage from repeated removal and reapplication. *(Please follow your local advice/guidance on the use of dressings under masks).*

Adhesive Remover Wipes

Medical Adhesive Remover may be used to remove adhesive dressings without damaging or stripping the skin

- Gently lift the edge of the dressing
- Gently wipe underneath the dressing while gradually removing it from the skin

For further information or guidance,
contact Medicareplus International:

Phone: +44 (0)20 8810 8811
Email: info@medicareplus.co.uk
www.medicareplus.co.uk

Ordering Information

Product	Format	Selling Unit	Product Code	PIP Code	NHS-SC Code
MEDI DERMA-S Total Barrier Film	Wipe	30 Pack	60307	341-3184	ELY788
	Sterile Applicator	1ml - 5 Pack	61076	362-8716	ELY532

LIT907/MPL907/A4FSPAG/04.20