

AVAILABLE IN SPRAY AND TUBE

# USER GUIDE

# MEDI DERMA-PRO

Skin Protectant Ointment and Foam & Spray Incontinence  
Cleanser

TOTAL  
**BARRIER**  
PROTECTION™




# How to Use MEDI DERMA-PRO Skin Protectant Ointment



## Step 1:

- Clean and dry the skin.

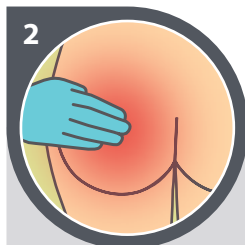


## Step 2:

- Gently apply a thin layer evenly over the affected area. Apply more on very wet or damaged skin.



## Step 3:

- If you have been given another treatment to use at the same time, let the first soak in for 5 minutes before applying **MEDI DERMA-PRO** Skin Protectant Ointment

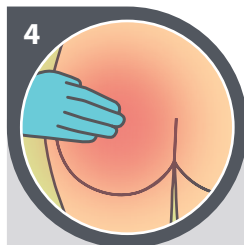

## Step 4:

- Clean the area and reapply after each episode of incontinence or when the skin no longer looks shiny.



### Skin Protectant Ointment:

For **EXTERNAL** use only. Do **NOT** use on deep wounds or deep skin damage. Do **NOT** use if there are signs of skin reaction following product application, or if there are known allergies/sensitivities to any of the ingredients. Consult your clinician accordingly. This product should **NOT** be used with adhesive dressings or appliances as they will not adhere to the ointment. Skin should be kept visible so improvements or deteriorations can be seen. Seek advice from your healthcare professional if you are concerned. Only use as directed. Do **NOT** use past expiry date or if product is damaged. Once opened, product can be used through to expiry date. For single patient use only. Keep out of reach of children.


# How to Use MEDI DERMA-PRO Foam & Spray Incontinence Cleanser

PART OF OUR  
SKIN PROTECTANTS  
& WOUND  
CARE RANGE



## Step 1:

- Turn nozzle to (1).
- **Foam Mode:** Keep flap closed and pull lever.
- **Spray Mode:** Open flap (2) and pull lever.

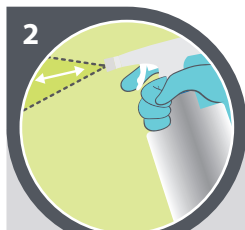


## Step 2:

- Apply directly to the skin or onto a wash cloth.



## Step 3:

- Gently clean the area. Try not to rub. No need to rinse.

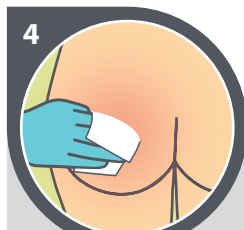

## Step 4:

- Pat dry or leave to dry in the air if possible.



### Foam & Spray Incontinence Cleanser:

For **EXTERNAL** use only. Do **NOT** use if there are signs of skin reaction following product application, or if there are known allergies/sensitivities to any of the ingredients. Consult your clinician accordingly. Only use as directed. This Product may affect the adherence of some adhesive dressings and devices. Do **NOT** use past expiry date or if product is damaged. Once opened, product can be used through to expiry date. For single patient use only. Keep out of reach of children.



SCAN HERE For more  
information on **MEDI DERMA-PRO**