

**Components Comparison
Against Competitor Packs:**

Nurse It® - Pack Contents		Dressit*	Softdrape*	Polyfield**
Non-Woven Swab	● (x7)	● (x4)	● (x5)	● (x7)
Laminated Paper Sterile Field	● (x2)	● (x1)	● (x1)	● (x1)
Paper Measuring Tape	● (20cm)	● (14cm)	● (14cm)	● (10cm)
Compartment Tray	●	●	●	○
Disposable Forceps	●	○	○	○
Large Apron	●	●	●	●
Pair of Latex Free Gloves	● (Wrapped)	●	●	●
Paper Towel	●	●	●	●
White Polythene Disposal Bag	●	●	●	●
Absorbent Pad	○	●	○	○

Nurse It® is a registered trademark of Medicareplus International Limited.
*Dressit and Softdrape are registered trademarks of Richardson Healthcare Limited.
**Polyfield is a registered trademark of 365 Healthcare Limited.



Medicareplus
INTERNATIONAL Est. 2008

Medicareplus International Limited
Chemilines House | Alperton Lane | London | HA0 1DX | United Kingdom
Telephone: +44 (0)20 8810 8811 | Email: info@medicareplus.co.uk
www.medicareplus.co.uk



Nurse It®
Sterile Dressing Pack

Ordering Information:

Product Code	PIP Code	Pack Quantity	Bunzl Code	NHS-SC Code
--------------	----------	---------------	------------	-------------

Nurse It®
with Small/Medium Nitrile Gloves



60154	351-2407	10	MB60154	EVS724
-------	----------	----	---------	--------

Nurse It®
with Medium/Large Nitrile Gloves



60239	336-9170	10	MB60239	EVS725
-------	----------	----	---------	--------



LIT063/MP/LB23/NI/ASBROCHURE/06.20

Nurse It® Sterile Dressing Pack

The Complete Sterile Dressing Pack Solution

Complete Solution

- Comprehensive range of components to meet a clinician's needs when changing a patient's dressing or performing additional aseptic techniques
- Unlike other products in the market, contains disposable forceps, a compartment tray and wrapped gloves
- Available in two variants - Small/Medium Gloves and Medium/Large Gloves

Providing Quality and Protection

- Clinicians have commented on the good quality and durability of the components compared to other dressing packs
- A large apron and easy-to-don nitrile gloves protect the clinician and patient during the procedure
- Laminated paper sterile fields help provide a sterile environment whilst changing a patient's dressing

Providing Cost Savings

- Competitively priced to provide a cost-effective solution for your medical facility

Large Apron

To protect clinician and patient whilst cleansing and dressing the patient's wound.

Polythene Disposal Bag

To collect soiled materials. Can be sealed using the adhesive strip.

Nitrile Gloves

Latex free, powder free gloves folded back in memory paper for ease of aseptic donning.

Laminated Paper Sterile Fields

2 Pieces that can be used to place pack components on and place under the patient's wound area.

Non-Woven Swabs

7 Pieces to aid cleansing of the wound area.

Compartment Tray

For holding solution, e.g. saline.

Paper Towel

Can be used to soak up solution whilst cleansing a patient's wound.

Paper Measuring Tape

20cm in length with space for recording patient and wound details.

Disposable Forceps

Can be used to pick up and soak items such as swabs in saline solution.

How to Use Nurse It® Using Aseptic Technique:

Please also refer to [YouTube](#) video:

"Nurse It - How to Use Guide - Aseptic Technique."

Action	Description
Wash Hands	Use soap and water or cleansing wipes/alcohol gel
Prepare Work Surface	Decontaminate with a detergent wipe, dry the area and then disinfect with a 70% alcohol wipe
Assemble and Organise Equipment	Nurse It® Dressing Pack and the other items required to dress the wound
Open Nurse It® Dressing Pack	Place the pack on a clean work surface area and spread out using the corners of the Sterile Field to prevent contamination
Apply Apron	The Apron's large surface area protects the clinician from any splash-back or contaminated equipment and prevents cross-contamination
Arrange Equipment	Arrange and remove the equipment using aseptic non-touch technique
Prepare Wound Cleansing	Fill the Compartment Tray with preferred cleansing solution
Decontaminate Hands	Use an alcohol gel
Remove Old Dressing	Wearing some non-sterile gloves, put the Paper Towel underneath the wound and remove the old dressing, placing it in the Disposal Bag
Cleanse Wound	Use moistened Non-Woven Swabs to cleanse the wound, then dry around wound area. Remove gloves and dispose
Decontaminate Hands	Use soap and water or cleansing wipes/alcohol gel
Redress the Wound	Apply the Nitrile Gloves from the Nurse It® Pack. Measure and record the size of the wound with the Paper Measuring Tape , then apply a new dressing
Dispose of Equipment	Remove the Apron and dispose of all equipment in the Disposal Bag . Clean the work surface with a detergent wipe
Wash Hands	Use soap and water or cleansing wipes/alcohol gel

